



E L E

N K

E R

HFK-9213-5

ROLLATOR
WALKER

HFK-9213-5

ROLLATOR
WALKER



USER MANUAL

Parts:



- | | |
|---------------|--------------|
| 1 Backrest | 6 Handgrip |
| 2 Armrest Pad | 7 Handbrake |
| 3 Storage Bag | 8 Seat |
| 4 Foot Rest | 9 Rear Wheel |
| 5 Front Wheel | |

SET-UP INSTRUCTIONS

STEP 1

Unbox the walker on a level floor.



ACCESSORIES LIST



x4 Knobs



x2 Bolts and Nuts



x1 Cane Holder



x1 Cup Holder



x1 Safety Belt

STEP 2


Four Wheels Assembly

Take out the main frame. (fig 2).




Rear Wheels Assembly

Press the spring button of the wheel tube, then insert the rear wheel tube into the rear main frame tube. (fig 2-3. 2-4) There are labels on tube, "L1-L1, R1-R1". Then insert the knob to secure the rear wheel tube in place.(fig 2-5. 2-6. 2-7)

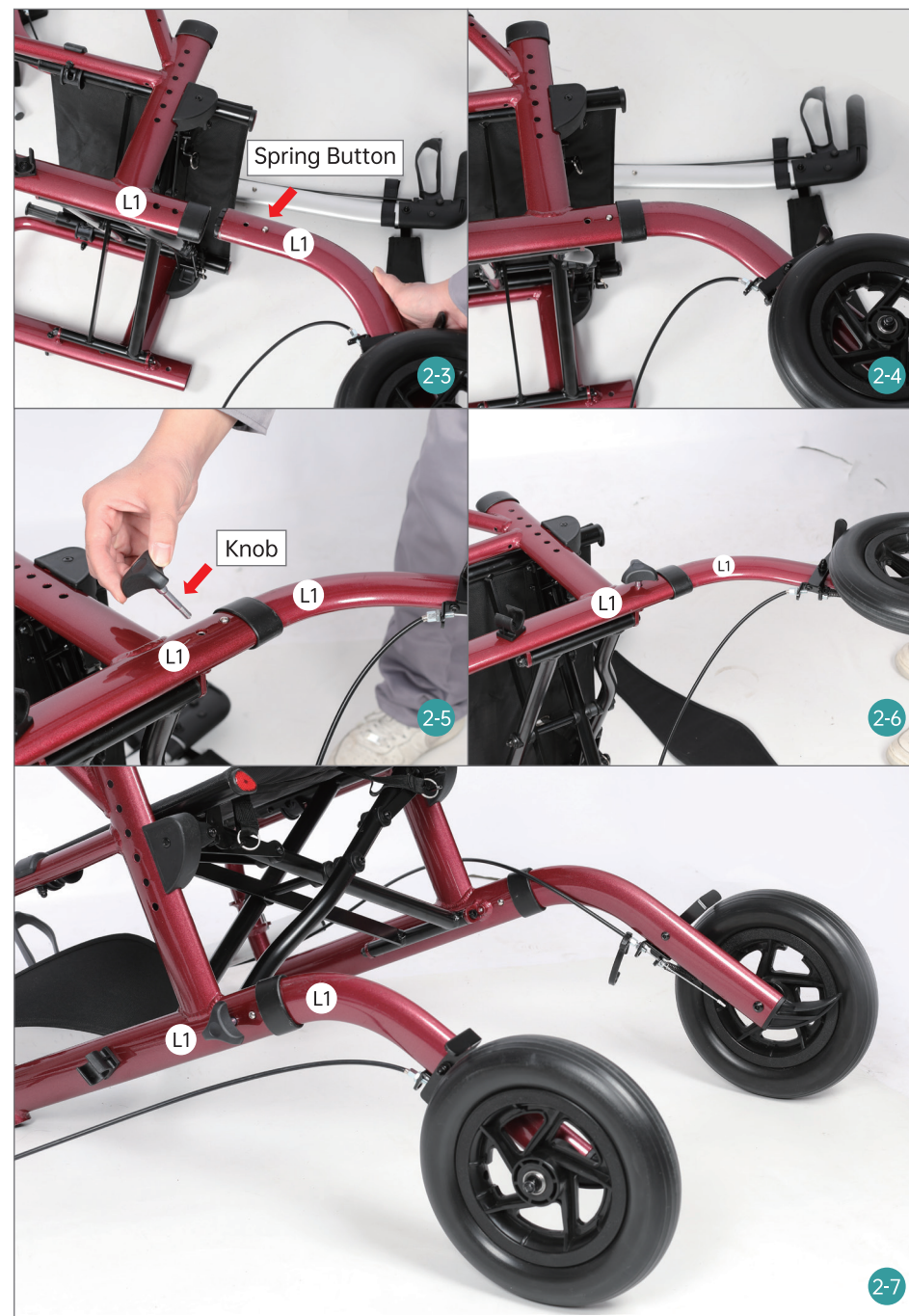

Front Wheels Assembly

Please take out the black plastic plug of front wheel tube before assembling the front wheel(fig 2-6, 2-7). Insert the front wheel tube into the front main frame tube. (fig 2-8) There are labels on tube, "L2-L2, R2-R2". Then insert the knob to secure the front wheel tube in place.(fig 2-9, 2-10, 2-11)

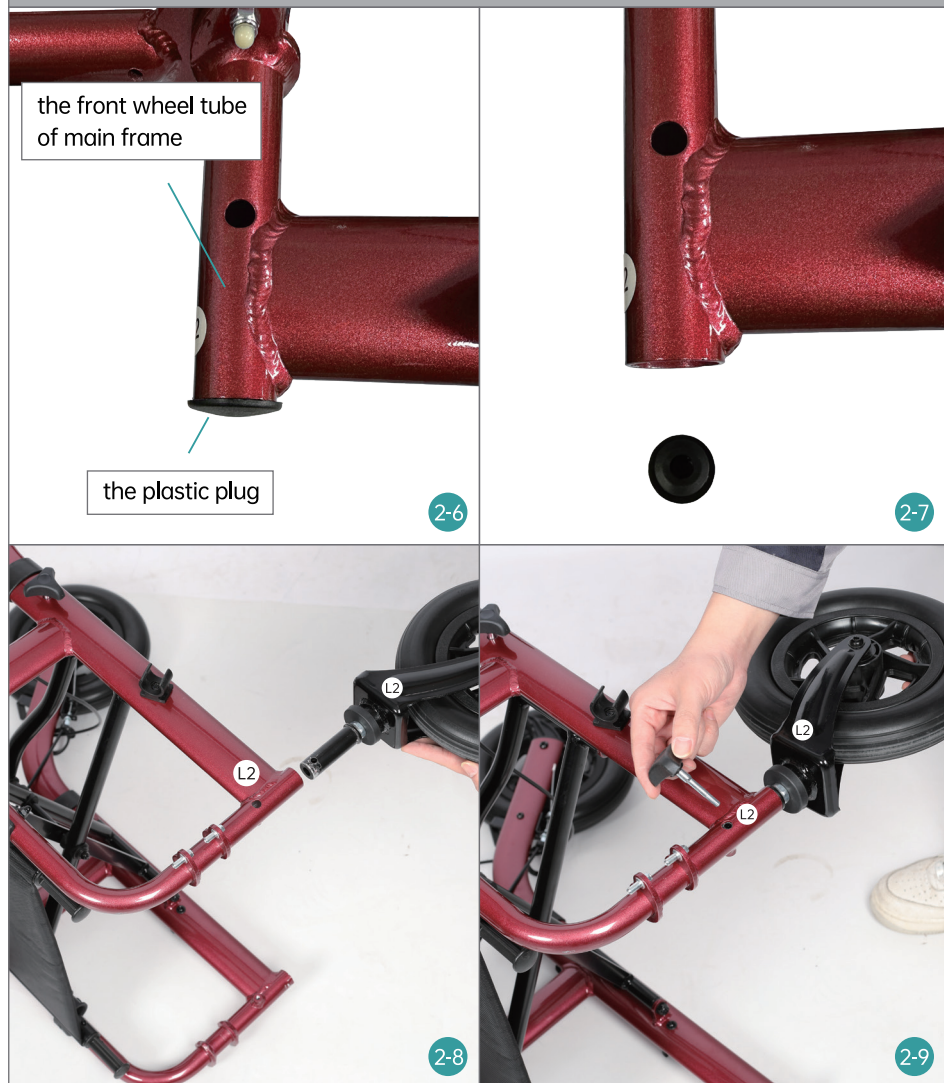

STEP 3 Handle Bar Assembly


Insert the handle bar, raise or lower the handle bar to the desired height.(fig 3, 3-1) There are labels on tube,"L3-L3, R3-R3".



STEP 4

The Footrest Assembly

Insert the footrest tube into the front main frame. (fig 4. 4-1) Then insert the footrest pad into the footrest tube.(fig 4-2. 4-3) There are labels on tubes "L4-L4, R4-R4-R4".

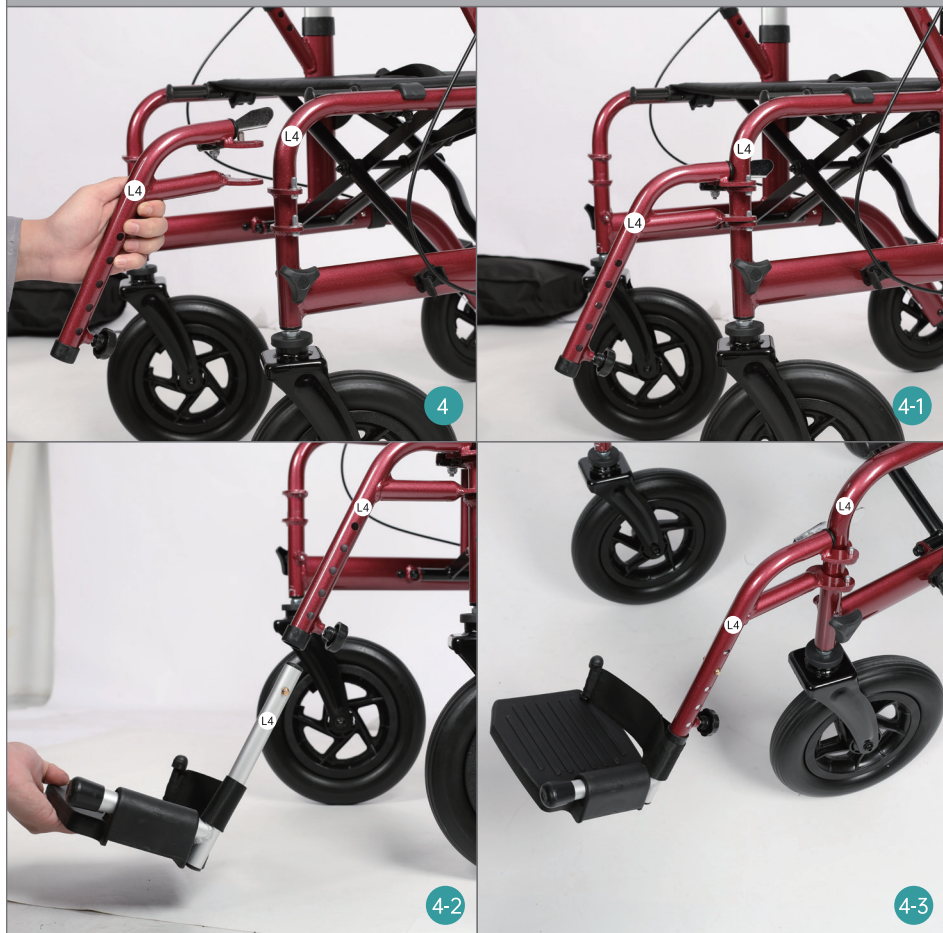

STEP 5

The Armrest Assembly


Insert the armrest into the main frame (fig 5), then secure the armrest with bolt and nut (fig 5-1. 5-2). There are labels on tubes "L5-L5, R5-R5".



STEP 6

Cane/Umbrella Holder Assembly

There are a cane knob on right side of the main frame, insert cane holder into it.(fig 6. 6-1)

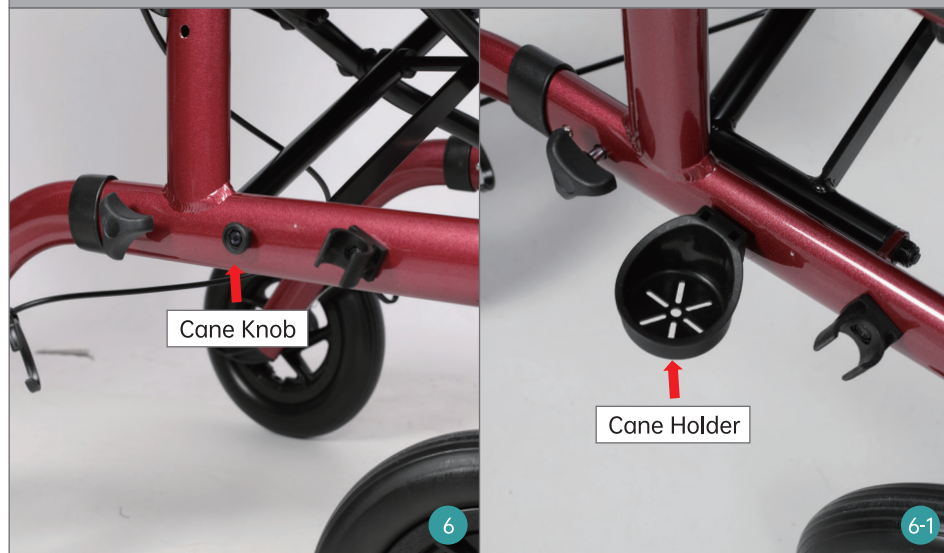

STEP 7

Storage Bag Assembly

Attach the bag on the front of main frame.(fig 7. 7-1)



STEP 8

Brake Cable Hook and Loop Fastener Assembly


Affix the brake cable to the main frame with hook and loop fastener(fig 8).



STEP 9

Safety Belt Assembly

Insert safety belt into handle bar, to secure person in place when sitting. (fig 9. 9-1. 9-2)

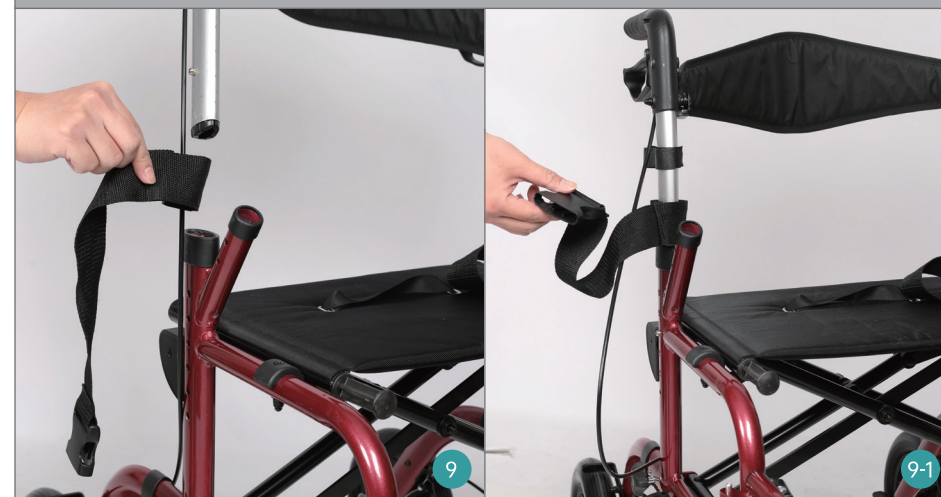



STEP 10 Cup Holder Assembly

Affix cup holder on left handle bar (fig 10).





Transform Rollator into Transport Wheelchair

Step 1: flip over the backrest
step 2: open the footrest



The Curb Climber

Use it to overcome obstacles such as curbs and thresholds.

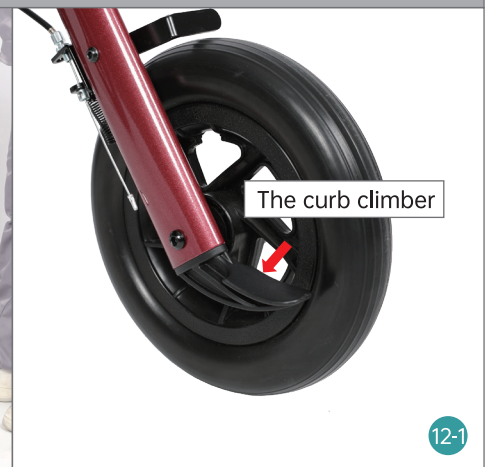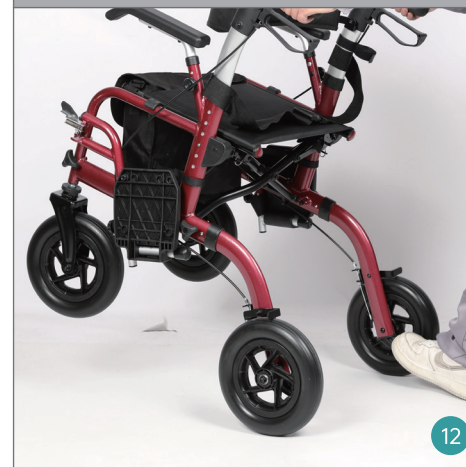

Handbrakes Operation

1. Squeeze to slow.
2. Push down to lock the rear wheels.



One-hand Folding

Pull up the strap of seat to fold the walker.



How to Adjust Brakes

Brake Issue: The brake cannot engage rear wheel when squeezing handbrake (fig A).



Adjust Rear Brake Position

Tools : Plier and 10mm wrench(or 8mm+10mm wrenches)(fig 1).
Use 10mm wrench to hold M6 nut, then use 8mm wrench or plier to turn M5 nut clockwise(fig 2-3). And brake will be closer to wheel(fig 4). Then squeeze handbrake, if it doesn't work, repeat the steps until rear brake can engage wheel.





CONTACT US

For any defects, parts missing, use or repair concerning our products, please contact us first, our customer service will reply within 24 hours and offer a good solution. Thank you for your support.






1. E-mail: elenkerservice@hotmail.com
2. Amazon Message: Click the Sold by "store name" under the buy box, then "ask a question". (Amazon Buyer)
3. Customer Help Line: (800) 401-7769

WARRANTY

- * You may return most new, unopened items sold and fulfilled by Amazon within **30** days of delivery for a full refund.
- * Defects in material accessories and non-durable components such as wheels, brakes, seat, handgrips and under NORMAL use for the **lifetime** warranty of the original consumer purchaser.
- * The warranty does not extend to device failure due to owner misuse or negligence, or normal wear, tear and need periodic replacement.
- * During warranty periods, our warranties cover all parts labor and shipping for repairing this walker.

WARNINGS

To ensure your safety in using ELENKER Rollator Walker, all the following instructions must be followed

-  The maximum load of the walker is 300lbs.
-  DO NOT use walker, if there is apparent damage or if any parts are missing.
-  Ensure that brakes, height adjustment bolts or any other moving parts are in good condition and check all screws and fasteners, and retighten if necessary.
-  DO NOT use on stairs, escalators, moving sidewalks, or go up or down inclines greater than 15% or diagonally across inclines greater than 5%.
-  Be careful to avoid pinching your fingers while raising and lowering the handle bar and opening and closing the rollator walker frame.