



E L E

IN K

E R

KNEE
WALKER



KNEE
WALKER

USER MANUAL OF ELENKER KNEE WALKER



Parts:



HOW TO INSTALL THE KNEE WALKER


STEP 1

Open the package, take the knee walker out, and remove protective foam.



STEP 2 Install Steering Column

Connect two receptacles together by inserting the folding lock through the aligned holes. Tighten the folding lock cap to secure two wheel parts in place.

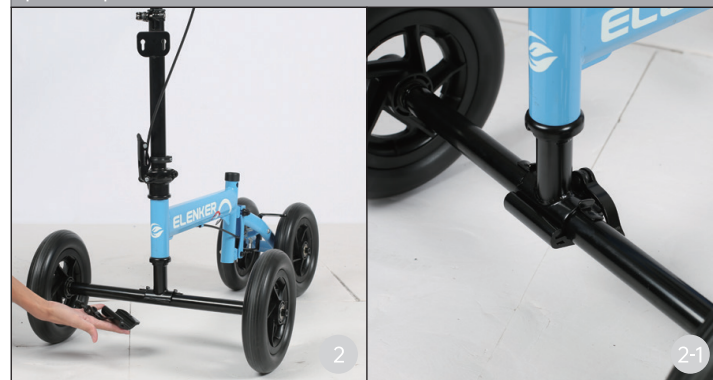

STEP 3

Install Knee Platform and Bag

Install the knee pad into the tube and insert the screw into the one of holes and tighten it.



⚠ WARNINGS

The knee walker can provide assistance to an individual weighing up to 300 lbs.

Precautions:

- ⚠ Do not expose the product to rain.
- ⚠ Please do not hang anything from the handlebars. This may cause the Knee Walker to tip, resulting in bodily injury or Knee Walker damage. Items should be placed in the basket.
- ⚠ Keep the product away from children, for they may tip over and injure themselves.
- ⚠ Be careful to avoid pinching your fingers while raising and lowering the handlebars and opening and closing the Knee Walker frame.
- ⚠ Do not use strong corrosive action cleaner onto product surface.
- ⚠ The maximum load of the knee walker is 136kg/300lbs



CONTACT US

For any inquiries or quality-related issues concerning our products, please send an email to elenkerservice@hotmail.com , or call (800) 401-7769. We will respond to you as soon as possible.