



KNEE
WALKER



E L E

IN K

E R

KNEE
WALKER

USER MANUAL OF ELENKER KNEE WALKER



Parts:



How to install the knee walker

STEP 1


As shown in the figure, unbox the knee walker and unfold the steering column into the upright position.





STEP 2

Install the knee walker body: Align and insert the connecting rod in the middle of the knee scooter.

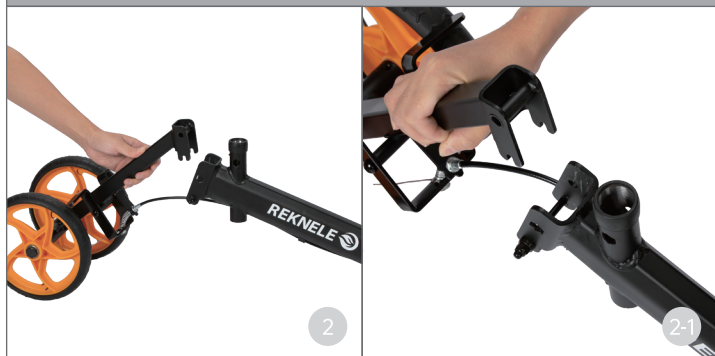

STEP 3

Insert a bolt (the longest one) and a knob into the rear frame and screw it securely. (The bolt head should be aligned to the opening, as shown below)



STEP 4

Install the front axle: insert the steering lever of the front axle, insert the bolt and screw it firmly.



STEP 5

Insert the handle bars into the steering column and tighten the screw.




STEP 6

Insert the knee pad into the tube and tighten the screw.




STEP 7

Fasten the brake cable with hoop and loop fastener.

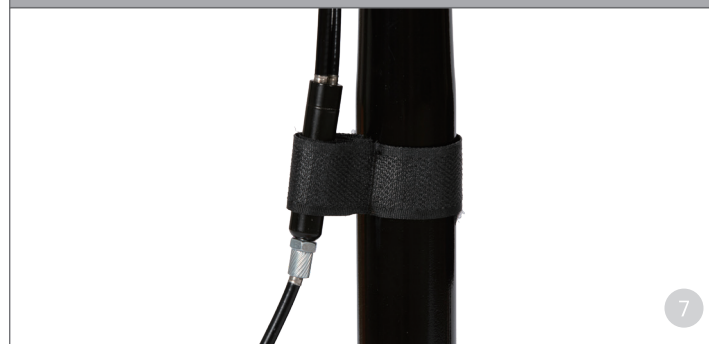

STEP 8

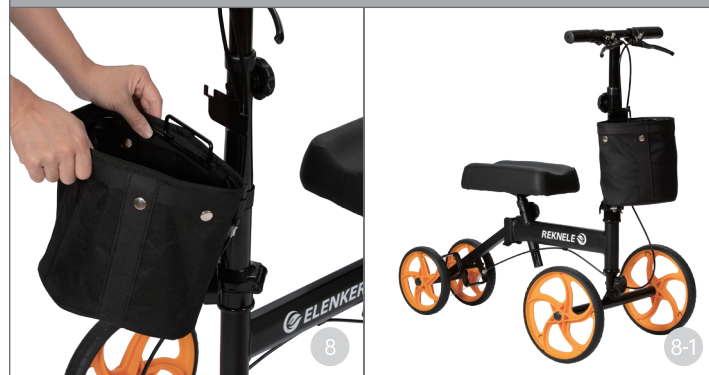


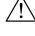
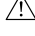
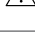
Insert the basket into the two grooves of the steering column, then push it down.



WARNINGS

The knee walker can provide assistance to an individual weighing up to 300 lbs.

Precautions:

-  Product shall be kept away from rain and in dry condition.
-  Please don't hang any things on both handlebars in order to prevent unstable movement of knee walker.
-  Keep the product away from children, so as to prevent tipping over and injury.
-  Care shall be taken during installation. Do not put fingers into the spin position.
-  Do not apply highly corrosive cleaner onto product surface.
-  The maximum load of the knee walker is 136kg/300lbs.

CONTACT US

For any inquiries or quality-related issues concerning our products, please send an email to elenkerservice@hotmail.com, or call (800) 401-7769. We will respond to you as soon as possible and give you an appropriate solution.