



KNEE
WALKER



E L E

IN K

E R

KNEE
WALKER

USER MANUAL OF ELENKER KNEE WALKER



Parts:



HOW TO INSTALL THE KNEE SCOOTER

STEP 1

As shown in the figure, unbox the knee scooter and unfold the steering column into the upright position.



STEP 2

Install the knee scooter body: Align and insert the connecting rod in the middle of the knee scooter.



STEP 3

Insert a bolt (the longest one) and a knob into the rear frame and screw it securely. (The bolt head should be aligned to the opening, as shown below)





STEP 4

Install the front axle: insert the steering lever of the front axle, insert the 3 bolts and screw it firmly.



STEP 5

Insert the handle bars into the steering column and tighten the screw.



STEP 6

Insert the knee pad into the tube and tighten the screw.



STEP 7

Insert the basket into the two grooves of the steering column, then push it down. Fasten the brake cable with hoop and loop fastener.



PARKING BRAKE

Simply squeeze brake handle firmly and press the button to engage the lock. To release the lock, squeeze the brake handle again.



This knee scooter can be folded.









Adjustable knee pad and handlebars to fit you well.



WARNINGS

The knee walker can provide assistance to an individual weighing up to 300 lbs.

Precautions:

-  Do not expose the product to rain.
-  Do not hang anything on either of the handlebars as this may cause the Knee Walker to tip.
-  Keep the product away from children, for they may tip over and injure themselves.
-  Please pay attention to safety while installing, do not put fingers into the spin position and seat cushion card buckled position.
-  Do not use strong corrosive action cleaner onto product surface.
-  The maximum load of the knee walker is 300lbs.

CONTACT US

For any inquiries or quality-related issues concerning our products, please send an email to elenkerservice@hotmail.com, or call (800) 401-7769. We will respond to you as soon as possible.