



KNEE
WALKER



E L E

IN K

E R

KNEE
WALKER

USER MANUAL OF ELENKER KNEE WALKER



Parts:










How to install the knee scooter

STEP 1




Accessories List

	4x Knobs		4x Bolts
	1x Shock absorber		1x Locking pin
	2x Release levers		1x Iron piece
	1x Bag		

STEP 2

Insert the shock absorber in the rear frame, insert and lock the release lever as the picture show. (Note: Please pay attention to the direction of the shock absorber, and the release lever is the longer one.)

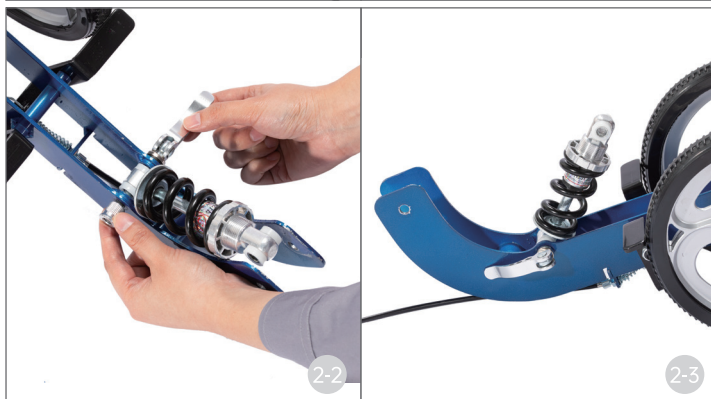

STEP 3

Raise the steering column on the front frame until the two mounting plates come together. Raise the clamp up and push it down firmly to the left or right (Double check it is firmly installed). Reverse the step when needing to lower steering column for transport.



STEP 4

Insert the locking pin through the aligned holes to connect the front frame.



STEP 5

Install the front wheel: Connect the two iron pieces of the front wheel, insert the shorter release lever and push it down to the left or right. Double-check it firmly installed.



STEP 6

Insert a bolt(the longest one) and a knob into the rear frame and screw it.




STEP 7

Insert the knee pad into the tube to the desired height. Insert a bolt into the hole and screw on knob to lock the knee pad in position.



STEP 8

Insert the tube into the frame. Line up holes to the desired height and insert a bolt into the hole. And screw on knob tightly to lock handlebar in position. (The bolt head should fit into the hex-shaped cut-out)

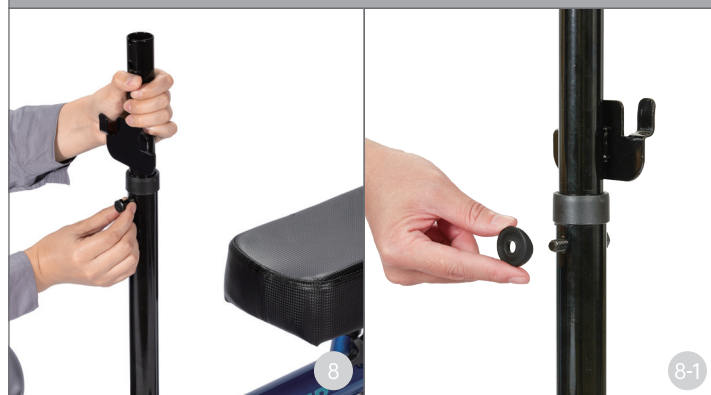



STEP 9

Insert the handlebar into the steering column. Insert a bolt into the hole and screw on knob tightly to lock handlebar in position.



STEP 10

Insert the bag into the two grooves of the steering column, then pushing down and tie the Velcro.



Parking brake

Simply squeeze hand brake and depress button to engage. To release the park brake, squeeze the hand brake again.









The steering column is foldable.



WARNINGS

The knee walker can provide assistance to an individual weighing up to 300 lbs.

Precautions:

-  Do not expose the product to rain.
-  Please do not hang anything from the handlebars. This may cause the knee scooter to tip, resulting in bodily injury or Knee Scooter damage. Items should be placed in the basket.
-  Keep the product away from children, for they may tip over and injure themselves.
-  Be careful to avoid pinching your fingers while raising and lowering the handlebars and opening and closing the knee scooter frame.
-  Do not use strong corrosive action cleaner onto product surface.
-  The maximum load of the knee scooter is 136kg/300lbs

CONTACT US

For any inquiries or quality-related issues concerning our products, please send an email to elenkerservice@hotmail.com , or call (800) 401-7769. We will respond to you as soon as possible.