




















KNEE WALKER

### **USER MANUAL OF ELENKER KNEE WALKER**



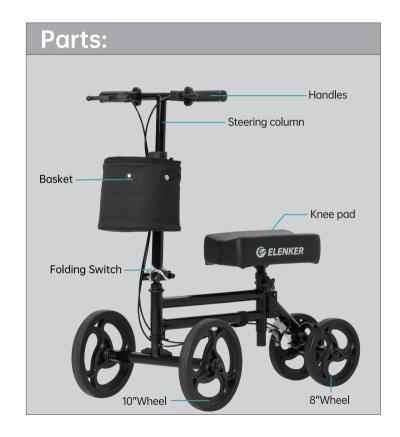

#### HOW TO INSTALL THE KNEE WALKER



#### STEP 1


Raise the steering column and fasten the clamp firmly to secure the steering column as shown in the figure.

#### STEP 2

Assemble the front wheels, insert the steering column into the front axle; insert the screw and fasten it with hex wrench and then tighten another retaining screw.




#### STEP 3

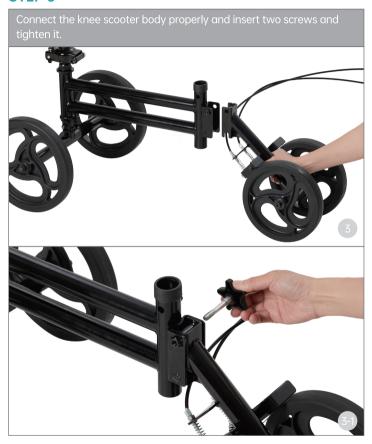

#### STEP 4




#### STEP 5

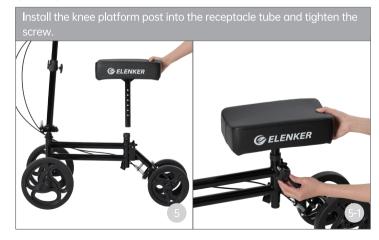

#### STEP 6



#### STEP 7

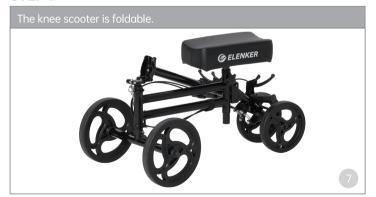

#### STEP 8



#### How to use the brake lock

circled in the figure to release the brake lock.

## **MARNINGS**

The knee walker can provide assistance to an individual weighing up to 300 lbs.

### **Precautions:**

- ⚠ Do not expose the product to rain.
- ① Do not hang anything on either of the handlebars as this may cause the Knee Walker to tip.
- ★ Keep the product away from children, for they may tip over and injure themselves.
- Please pay attention to safety while installing, do not put fingers into the spin position and seat cushion card buckled position.
- ⚠ Do not use strong corrosive action cleaner onto product surface.
- ↑ The maximum load of the knee walker is 300lbs.

# **© CONTACT US**

For any inquiries or quality-related issues concerning our products, please send an email to elenkerservice@ hotmail.com, or call (800) 401-7769. We will respond to you as soon as possible.