



E L E

N K

E R

YF-9007F  
ROLLATOR  
WALKER



YF-9007F  
ROLLATOR  
WALKER



# USER MANUAL

## Parts:



- |   |                   |    |  |
|---|-------------------|----|--|
| 1 | Handgrip          | 7  | Open Ends Hook and Loop Strap          |
| 2 | Cup Holder        | 8  | Hook and Loop Strap for<br>Brake Cable |
| 3 | Backrest          | 9  | Padded Seat                            |
| 4 | Under-seat Basket | 10 | Cane Holder                            |
| 5 | Front Wheel       | 11 | Rear Wheel                             |
| 6 | Handbrake         |    |  |

## SET-UP INSTRUCTIONS

## STEP 1

Unbox the walker on a level floor.



## Accessories



1x Cane holder



1x Cup holder



1x Open ends hook and loop strap to secure the upper part of cane



2x Hook and loop straps for brake cables



2x Bolts and nuts

## STEP 2

### Seat Assembly


Place the seat on the main frame with seat clip, there are two anti-slip stoppers to prevent the seat sliding, please make sure the clips are on the outside of the anti-slip stoppers(fig 2, 2-1, 2-2).



## STEP 3

### Front Wheels Assembly

Press spring button of wheel's leg, then insert it into the main frame(fig 3,3-1). The same steps to assemble right front wheel's leg(there are labels on legs and the main frame "L1-L1 R1-R1")(fig 3-2).

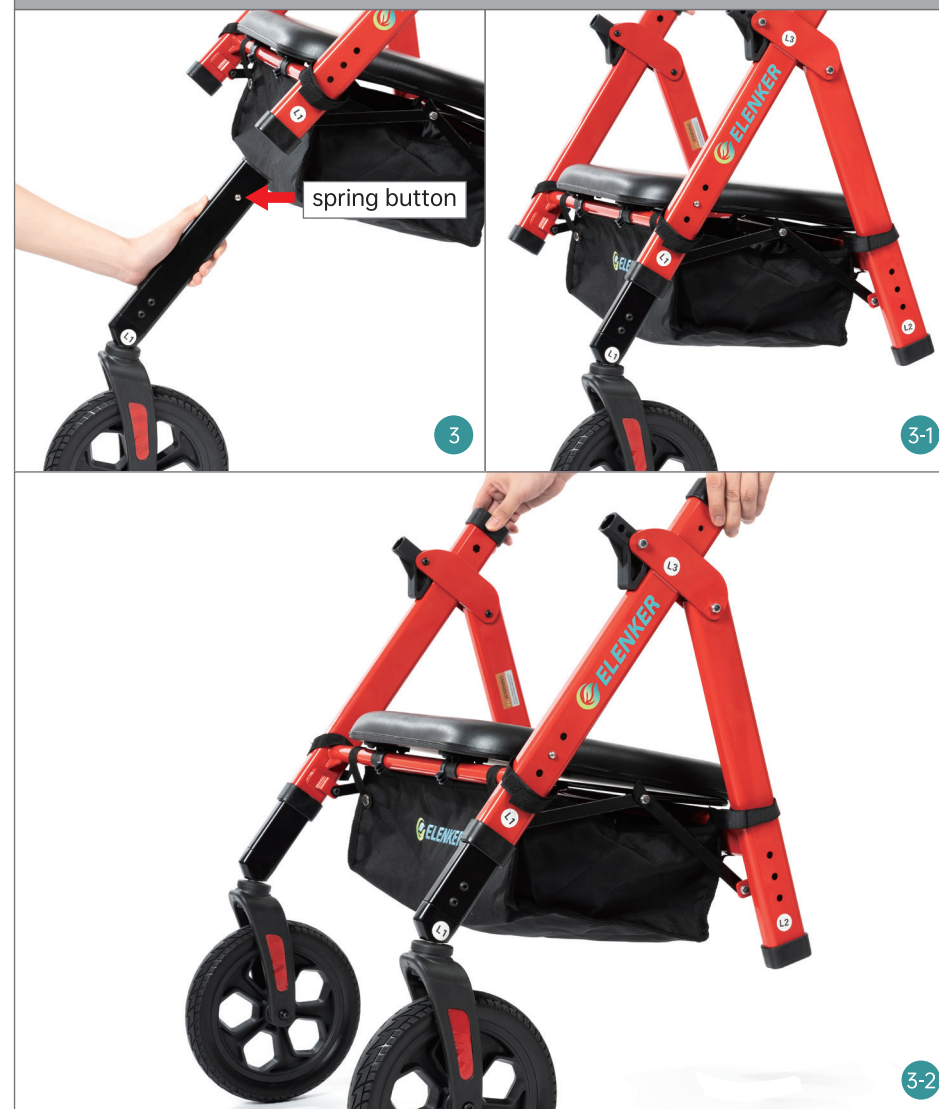

## STEP 4

### Rear Wheels Assembly


Press spring button of wheel's leg, then insert it into the main frame(fig 4,4-1). The same step to assemble right rear wheel's leg(there are labels on legs and the main frame "L2-L2 R2-R2")(fig 4-2).



## STEP 5

### Handle Bar Assembly

Insert the handle bar tube into the main frame(fig 5), then secure it with bolt and nut(fig 5-1 5-2). The same step to assemble right handle bar tube(there are labels on tubes and the main frame "L3-L3 R3-R3")(fig 5-3).

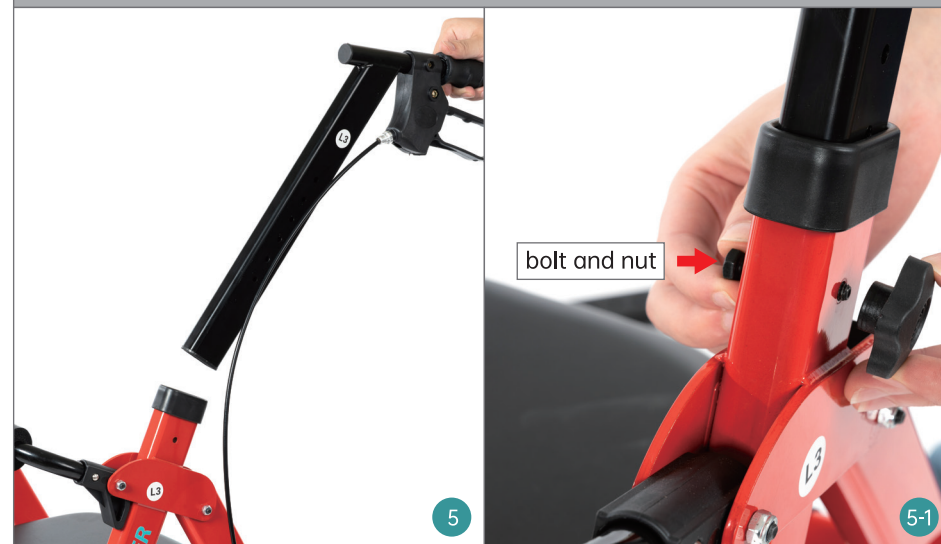



## STEP 6

### Backrest Assembly


Insert backrest into the main frame(fig 6, 6-1).



## STEP 7

### Brake Cable Hook and Loop Strap Assembly


Affix the brake cable to the main frame with hook and loop strap(fig 7).

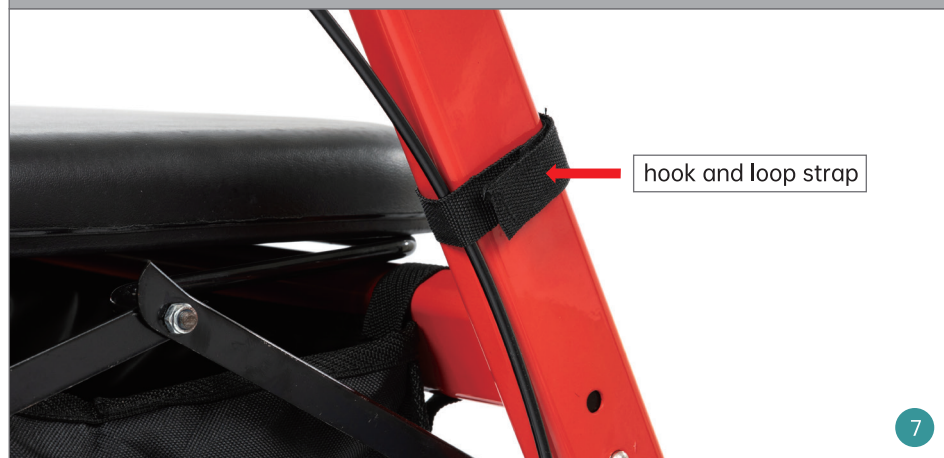

## STEP 8

### Cane/Umbrella Holder Assembly


Insert cane holder into cane holder knob (cane holder knob is on the left leg tube)(fig 8, 8-1). Then affix the open ends hook and loop strap for securing the upper part of cane(fig 8-2).

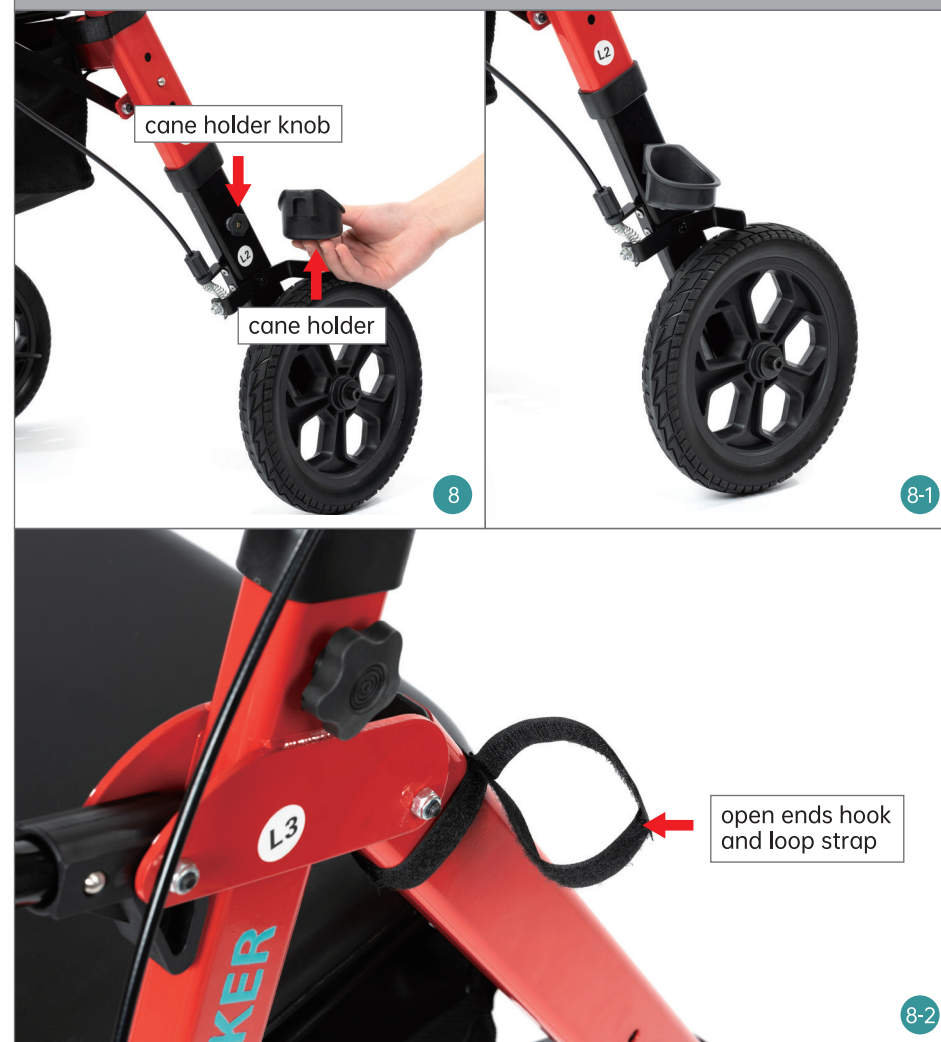

## STEP 9

### Cup Holder Assembly

There is a cup knob on the handle bar(fig 9), insert cup holder into cup knob (fig 9-1) .

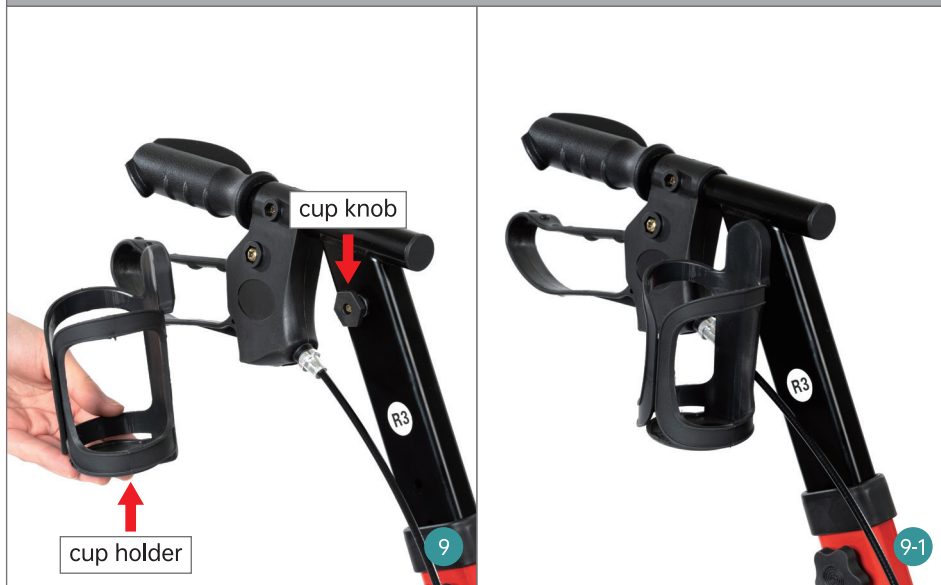

## The Curb Climber

Use it to overcome obstacles such as curbs and thresholds.



## HANDBRAKES OPERATION

1. Squeeze to slow
2. Push down to lock the rear wheels



## ONE-HAND FOLDING

Pull up the under-seat bracket to fold the walker.




## HOW TO ADJUST BRAKES


There are two ways to fix it.

**A: Adjust the screw housing.**


1. Press the handgrip down (fig 1,2).

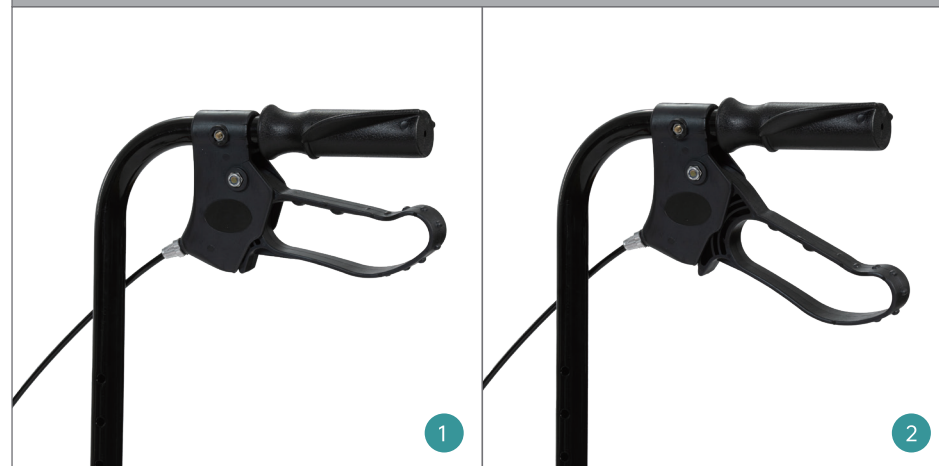

2. The rear brake cannot engage rear wheel when pressing the handgrip down. (fig 3);  
3. One hand hold the screw holder, and turn the screw housing left or right (fig 4,5) until rear brake can engage the rear wheel (fig 6).

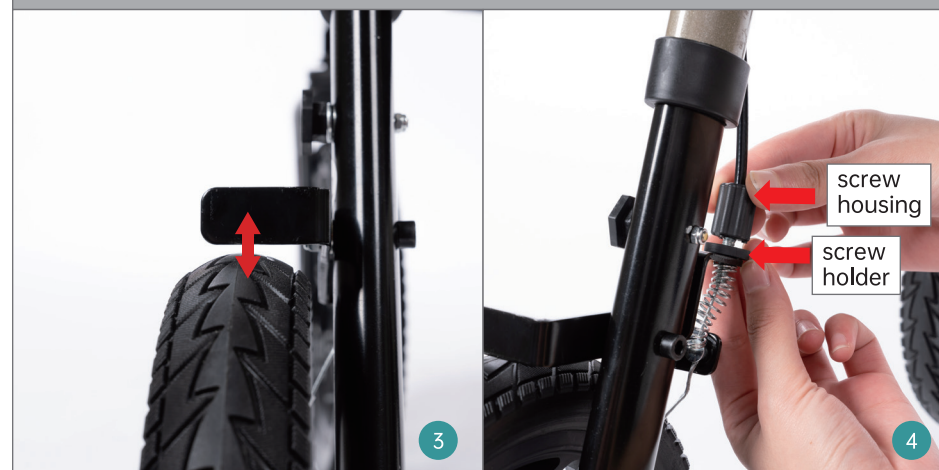




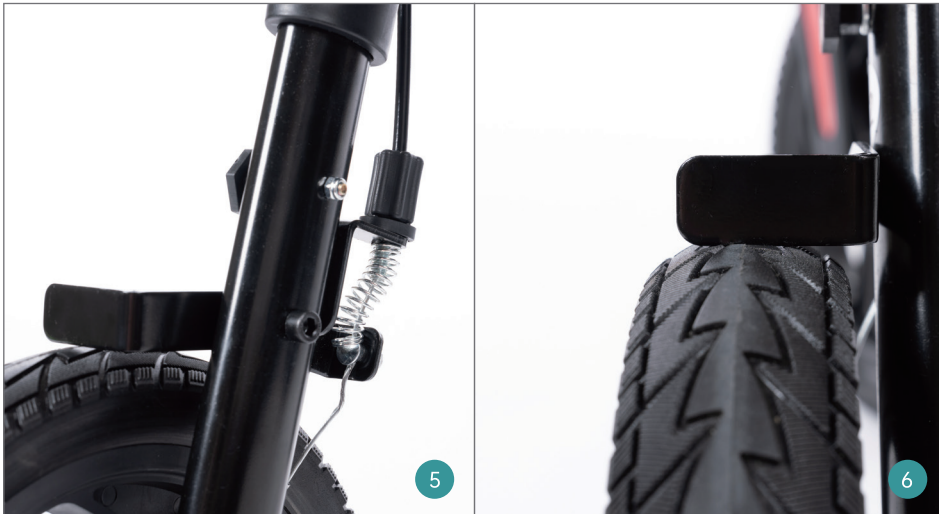

**B: If method one doesn't work, please tighten the brake cable.**

1. Press the handgrip down (fig 1,2).

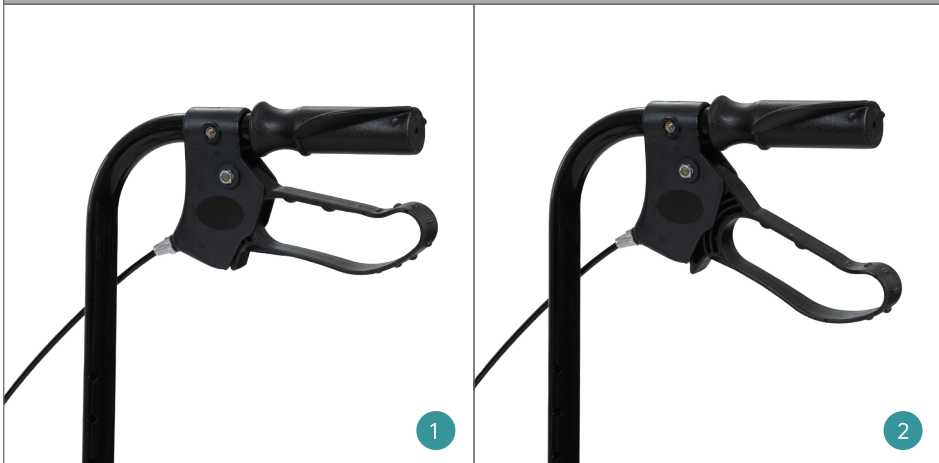

2. Loosen the screw, the spring will become loose, and the rear brake will loosen;  
 3. Use pliers to pull the brake cable while bringing the rear brake close to the wheel, then tighten the screw;  
 4. Put the handbrake to its original position, the wheel should be able to roll. Press the handbrake down or squeeze the handbrake, and the rear brake should come close to the wheel, indicating a successful adjustment.











## WARRANTY

- \* You may return most new, unopened items sold within 30 days of delivery for a full refund.
- \* Defects in material accessories and non-durable components such as wheels, brakes, seat, and handgrips under normal use are covered for the 3-year warranty of the original consumer purchaser.
- \* The warranty does not extend to device failure due to owner misuse or negligence, or normal wear, tear and need periodic replacement.

## WARNINGS

To ensure your safety in using ELENKER Rollator Walker, all the following instructions must be followed.

-  The maximum load of the walker is 500lbs.
-  DO NOT use walker, if there is apparent damage or if any parts are missing.
-  Ensure that brakes, height adjustment bolts or any other moving parts are in good condition and check all screws and fasteners, and retighten if necessary.
-  DO NOT use on stairs, escalators, moving sidewalks, or go up or down inclines greater than 15% or diagonally across inclines greater than 5%.
-  Be careful to avoid pinching your fingers while raising and lowering the handle bar and opening and closing the rollator walker frame.
-  The rollator walker is intended to provide additional help and support, but is not a substitute for medical treatment or rehabilitation. Please use it correctly under the guidance of professionals, and keep the accessories properly to avoid misuse or damage.

## CONTACT US

For any defects, parts missing, use or repair concerning our products, please contact us first, our customer service will reply within 24 hours and offer a good solution. Thank you for your support.

1. E-mail: [elenkerservice@hotmail.com](mailto:elenkerservice@hotmail.com)
2. Amazon Message: Click the Sold by "store name" under the buy box, then "ask a question". (Amazon Buyer)
3. Customer Help Line: (800) 401-7769