



E L E

IN K

E R

YF-9008A
ROLLATOR
WALKER



YF-9008A
ROLLATOR
WALKER



USER MANUAL

Parts:




- | | |
|---------------------------------------|---|
| 1 Handbrake | 9 Armrest Pad |
| 2 Backrest | 10 Bolt and Nut for Handle Bar |
| 3 Cup Holder | 11 Sit-to-stand Handle Bar |
| 4 Hook and Loop Strap for Brake Cable | 12 Open Ends Hook and Loop Strap |
| 5 Padded Seat | 13 Bolt and Nut for Sit-to-stand Handle Bar |
| 6 Under-seat Basket | 14 Cane Holder |
| 7 Front Wheel | 15 Rear Wheel |
| 8 Handgrip | |

SET-UP INSTRUCTIONS

STEP 1

Unbox the walker on a level floor.

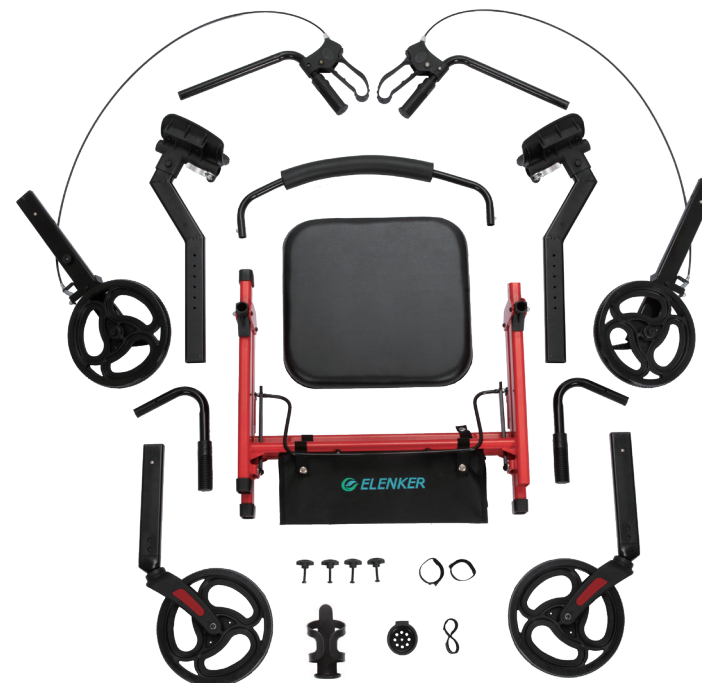


Accessories



1x Cane holder



1x Cup holder

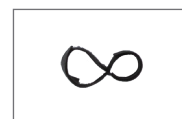

1x Open ends hook and loop strap to secure the upper part of cane



2x Hook and loop straps for brake cables



4x Bolts and Nuts (2 for sit-to-stand handle bars, 2 for handle bars)

STEP 2

Seat Assembly


Place the seat on the main frame with seat clip, there are two anti-slip stoppers to prevent the seat sliding, please make sure the clips are on the outside of the anti-slip stoppers(fig 2, 2-1, 2-2).



STEP 3

Front Wheels Assembly

Press spring button of wheel's leg, then insert it into the main frame(fig 3,3-1). The same steps to assemble right front wheel's leg(there are labels on legs and the main frame "L1-L1 R1-R1")(fig 3-2).

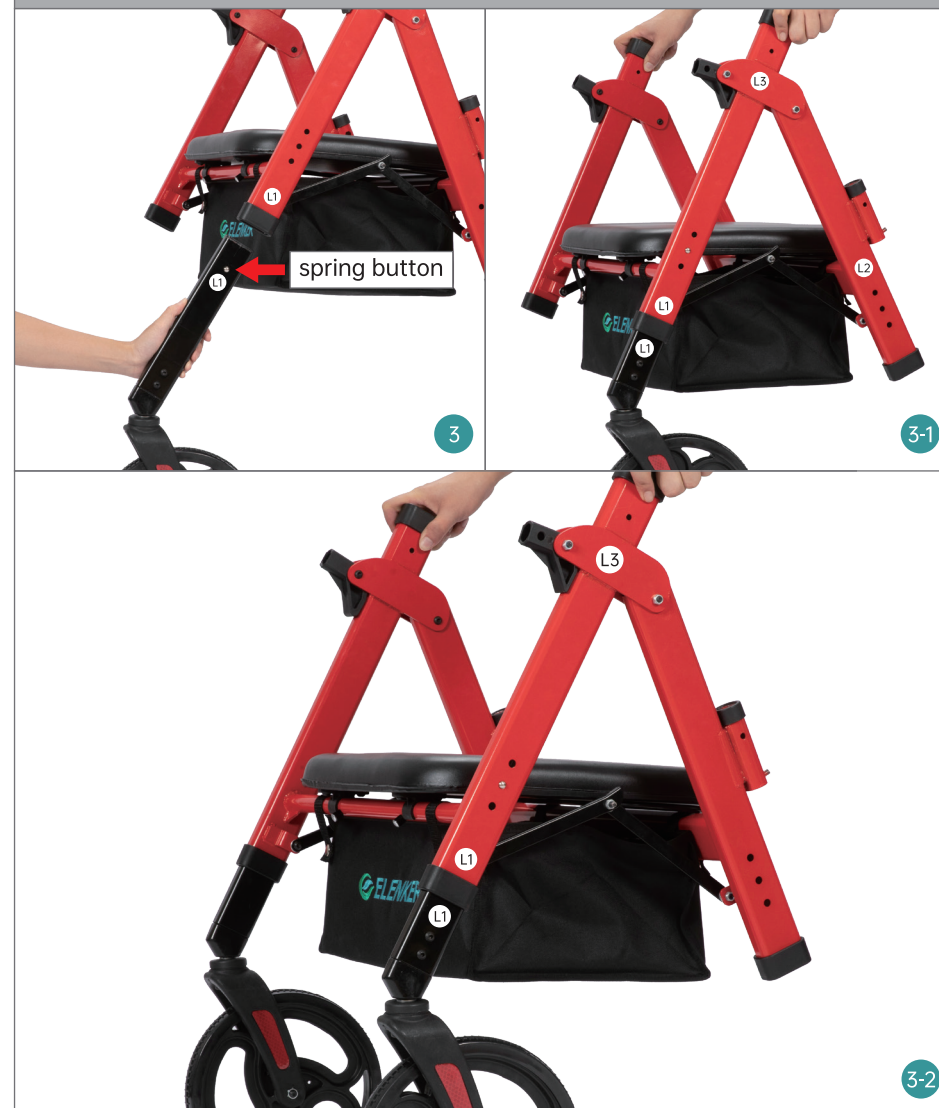

STEP 4

Rear Wheels Assembly

Press spring button of wheel's leg, then insert it into the main frame(fig 4,4-1). The same step to assemble right rear wheel's leg(there are labels on legs and the main frame "L2-L2 R2-R2")(fig 4-2).



STEP 5

Handle Bar Assembly


Insert the handle bar tube into the main frame(fig 5), then secure it with bolt and nut(fig 5-1 5-2). The same step to assemble right handle bar tube (there are labels on tubes and the main frame "L3-L3 R3-R3")(fig 5-3).



STEP 6

The Handgrip Assembly

Insert the handgrip tube into the armrest pad tube(fig 6), adjust length and angle to fit your forearm, then secure it by closing the handgrip clamp lever (fig 6-1, 6-2).

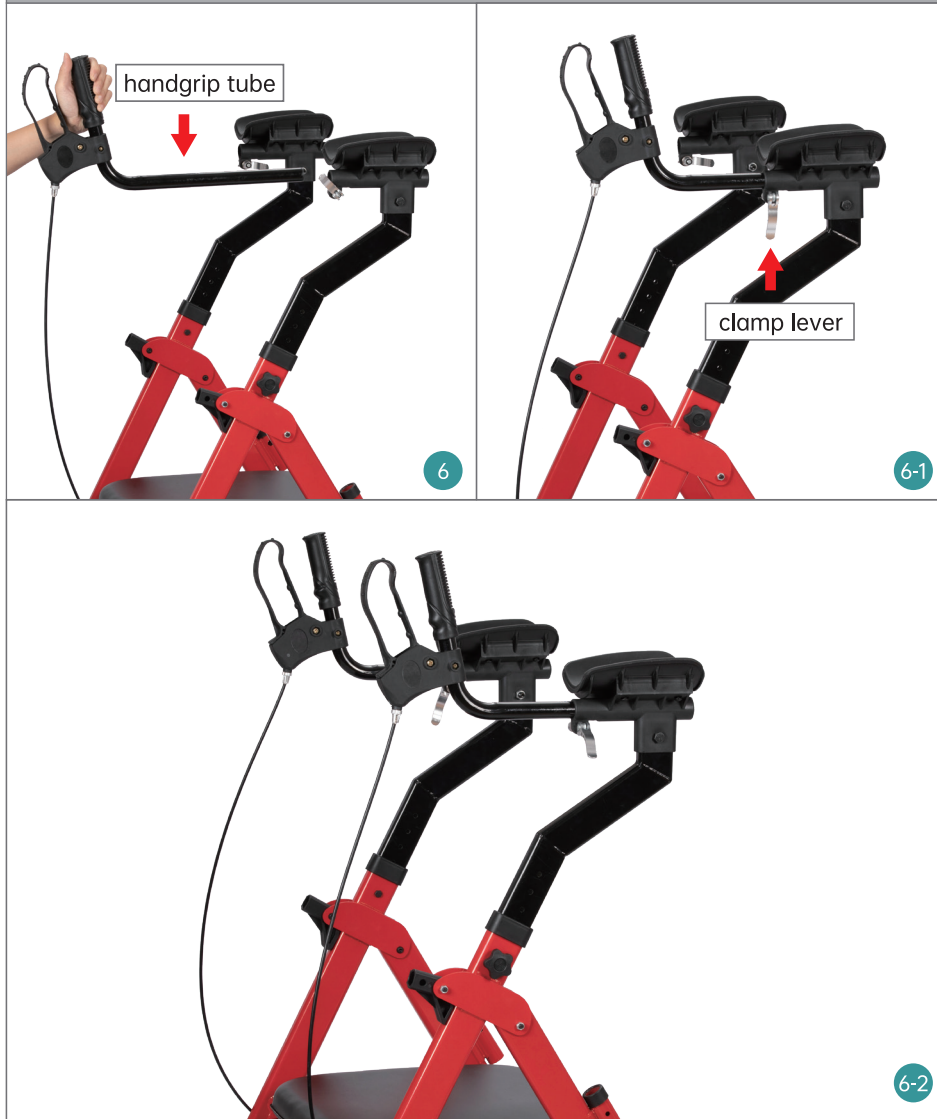

STEP 7

Backrest Assembly


Insert backrest into the main frame(fig 7, 7-1).



STEP 8

Sit-to-stand Handle Bar Assembly

Insert the sit-to-stand handle bar(fig 8), secure it with bolt and nut(fig 8-1, 8-2, 8-3).

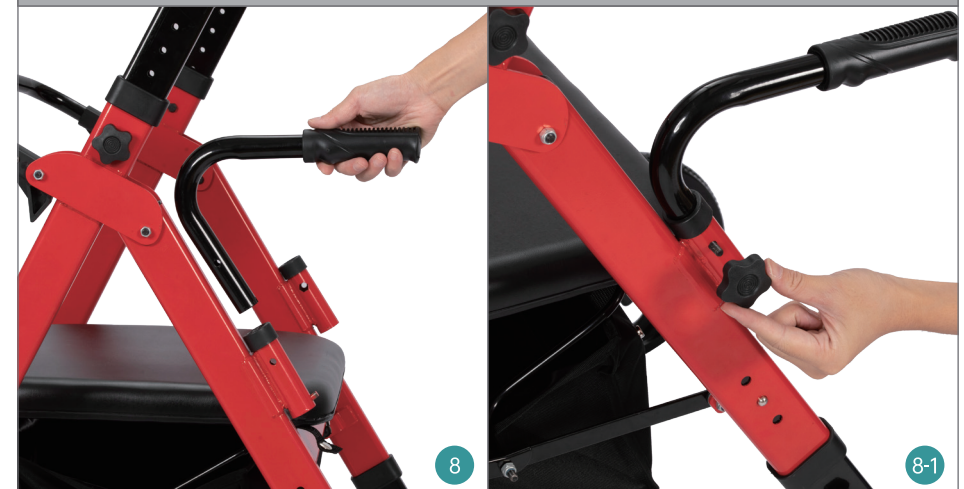




STEP 9 Brake Cable Hook and Loop Strap Assembly

Affix the brake cable to the main frame with hook and loop strap(fig 9).



STEP 10 Cane/Umbrella Holder Assembly


Insert cane holder into cane holder knob (cane holder knob is on the right leg tube)(fig 10, 10-1). Then affix the "8" shpae hook and loop strap for securing the upper part of cane(fig 10-2).

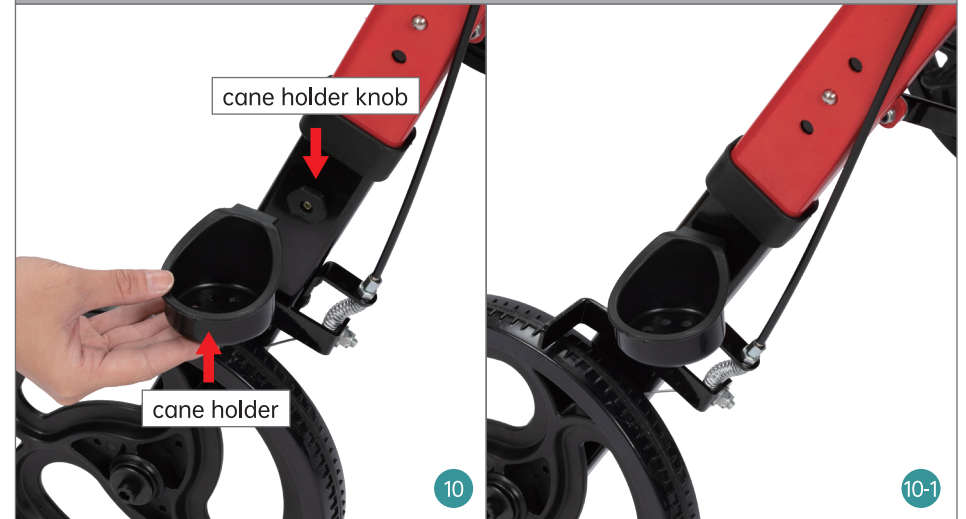

STEP 11

Cup Holder Assembly

There is a cup knob on the handle bar(fig 11), insert cup holder into cup knob (fig 11-1) .

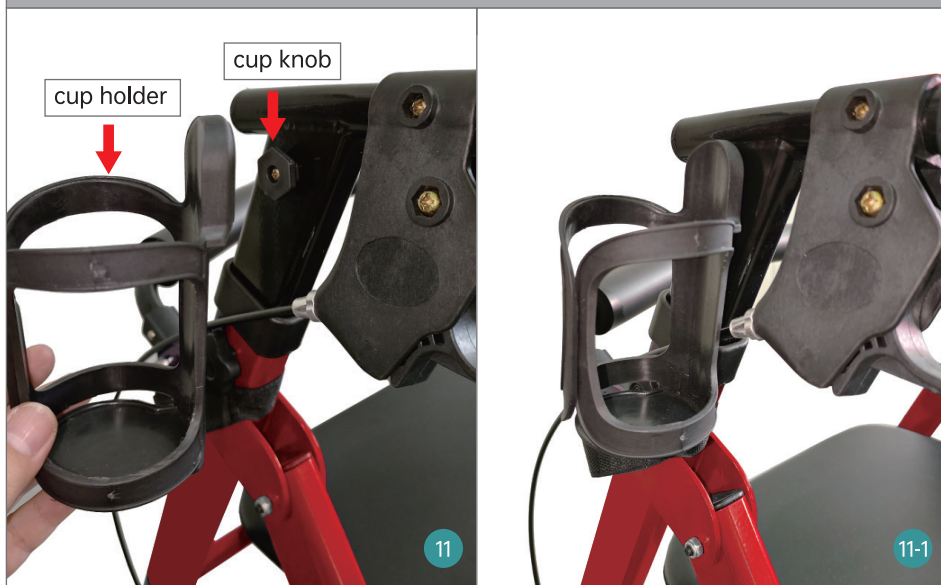

The Curb Climber

Use it to overcome obstacles such as curbs and thresholds.




HANDBRAKES OPERATION

1. Squeeze to slow
2. Push down to lock the rear wheels




ONE-HAND FOLDING

Pull up the under-seat bracket to fold the walker.

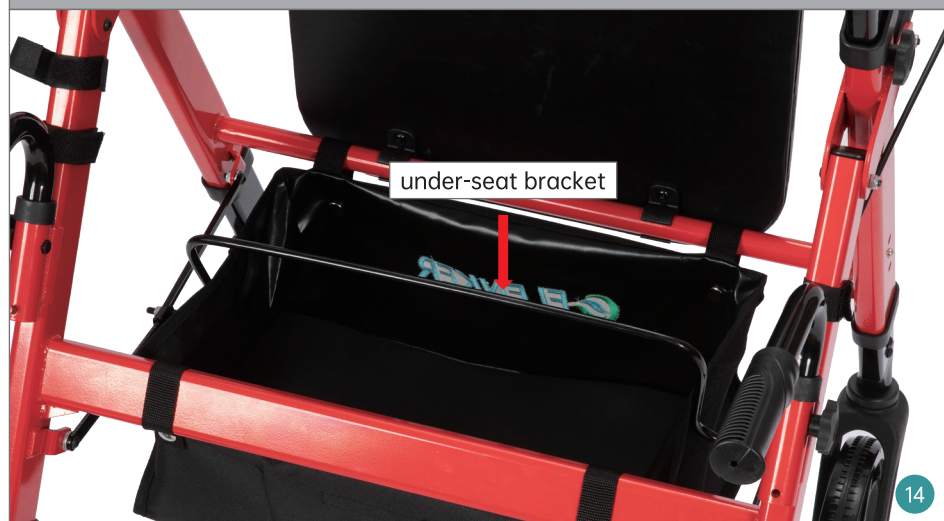


HOW TO ADJUST BRAKES

1. Press the handgrip down (fig 1,2)

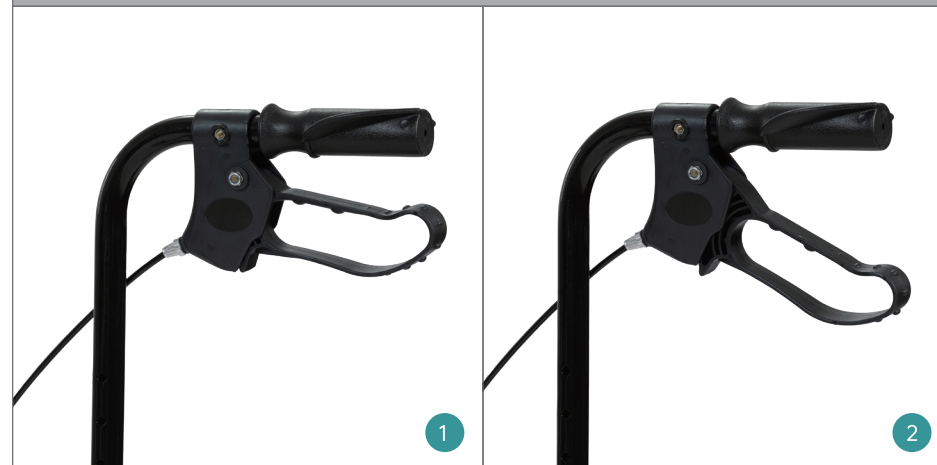

2. The rear brake cannot engage rear wheel when pressing the handgrip down. (fig 3)

3. One hand hold the screw holder, and turn the screw housing left or right (fig 4,5) until rear brake can engage the rear wheel (fig 6).

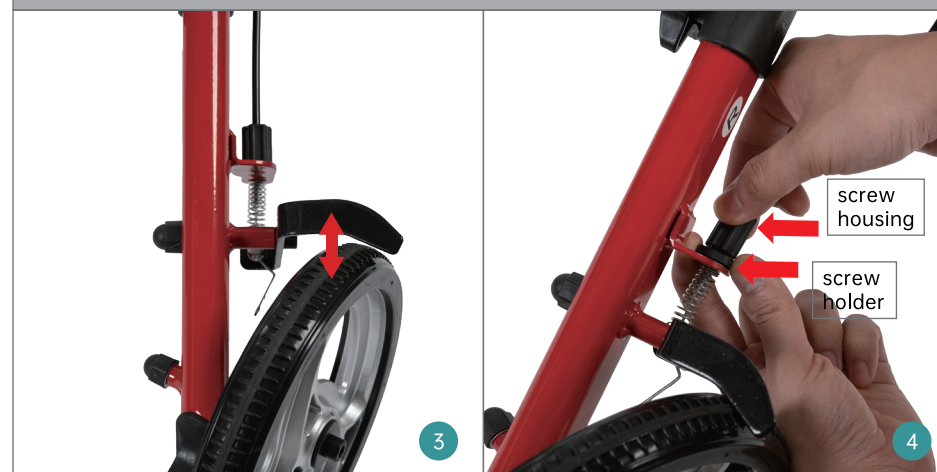









WARRANTY

- * You may return most new, unopened items sold and fulfilled by Amazon within **30** days of delivery for a full refund.
- * Defects in material accessories and non-durable components such as wheels, brakes, seat, handgrips and under **NORMAL** use for the **lifetime** warranty of the original consumer purchaser.
- * The warranty does not extend to device failure due to owner misuse or negligence, or normal wear, tear and need periodic replacement.
- * During warranty periods, our warranties cover **all** parts labor and shipping for repairing this walker.

WARNINGS

To ensure your safety in using ELENKER Rollator Walker, all the following instructions must be followed.

-  The maximum load of the walker is 500lbs.
-  DO NOT use walker, if there is apparent damage or if any parts are missing.
-  Ensure that brakes, height adjustment bolts or any other moving parts are in good condition and check all screws and fasteners, and retighten if necessary.
-  DO NOT use on stairs, escalators, moving sidewalks, or go up or down inclines greater than 15% or diagonally across inclines greater than 5%.
-  Be careful to avoid pinching your fingers while raising and lowering the handle bar and opening and closing the rollator walker frame.
-  The rollator walker is intended to provide additional help and support, but is not a substitute for medical treatment or rehabilitation. Please use it correctly under the guidance of professionals, and keep the accessories properly to avoid misuse or damage.

CONTACT US

For any defects, parts missing, use or repair concerning our products, please contact us first, our customer service will reply within 24 hours and offer a good solution. Thank you for your support.

1. E-mail: elenkerservice@hotmail.com
2. Amazon Message: Click the Sold by "store name" under the buy box, then "ask a question". (Amazon Buyer)
3. Customer Help Line: (800) 401-7769